

A new concept of polishing

The only automated system that provides better results than hand polishing



DLyte equipment automates metal alloy polishing for the dental sector, improving quality while reducing costs







Better results than manual polishing, in one step

The polishing action is based on a unique and innovative dry electropolishing technique that reaches every internal cavity which can not be accessed mechanically. DLyte allows the polishing of pieces from any casting, sintered or milled manufacturing processes.

DLyte offers high-quality polishing for cobalt-chrome and titanium.

Respects tolerances



Preserves the initial shape



Mirror finishing and homogeneous results



How it works



Step 1The parts are clamped onto the holder



Step 2
The holder is introduced into



Step 3The program is selected



Step 4
Average of 50-60 minutes per cycle







Impact on your business

REDUCE COSTS BY UP TO 80%

MAKE YOUR PRODUCTION MORE FLEXIBLE

STANDARDIZE YOUR PRODUCT



The dust emitted during manual polishing is dangerous for health.







Lower costs

The treatment does not require manual processing, reducing labor costs and eliminating the need to rework parts and the production of defective parts.

Time saving

DLyte is an automatic process which reduces the complexity of the current multi-step processes. The entire process is completed in one step.

Controlled production

DLyte offers a unique solution to meet the needs of the dental industry, with traceable and predictable results, according to polishing needs, production quantities, and/or piece dimensions.

Comparison between Traditional Polishing and the DLyte System

Removal of manufacturing Supports



ring cleaning of the items

TRADITIONAL POLISHING **METHOD**

DLYTE

POLISHING SYSTEM

Abrasive

Homogenization of the area and definition of angles.



Electrolytic Bath

Cleaning and metal shining.

Elimination of micro-

roughness. Standar-dization of hooks.

Polishing

Brightness

Polishing of the piece to give it a final shine.



Abrasive

Homogenization definition of angles.











These three processes have to be done with additive manufacturing in all production types.





Materials and applications

Thanks to the wide variety of dry electrolyte medias, DLyte offers a high-quality finishing for Cobalt chrome and Titanium pieces.

DLyte provides superior results for all types of fixed or removable dentures, implantology systems and orthodontic products.



Fixed prosthesis

Polishing process from 30 to 45 minutes (depending on the type of manufacturing)





Implant prosthesis

Polishing process from 30 to 45 minutes (depending on the type of manufacturing)









Removable prosthesis

Polishing process from 50 to 80 minutes (depending on the type of manufacturing)

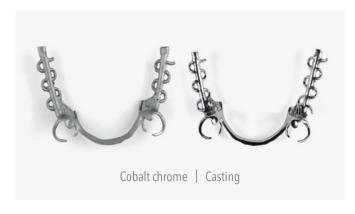

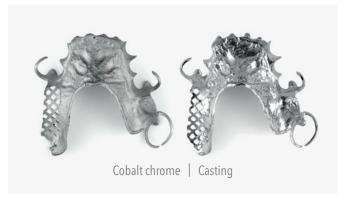

The results quarantee the effectiveness of our system



DLyte has proven the biocompatibility of the products processed with the DLyte System.

Proven biocompatibility

The medical and dental device sectors require clinically proven processes and products which ensure their compliance with the most demanding safety regulations.

The manufacturer must guarantee that the devices meet all appropriate requirements. In particular, they must perform a risk/benefit analysis and evaluate the biocompatibility and the toxicity of the materials used.

Corrosion resistance

The corrosion resistance of a piece of equipment is dependent not only on the selection of the most suitable alloy, but also on the correct treatment of the material. Many applications require a surface treatment after polishing to comply with corrosion resistance requirements. DLyte is a unique system that is able to remove roughness drastically and improve the corrosion resistance of the metal pieces while also reducing the number of processes required in the manufacturing process.



DLyte achieves better corrosion resistance than liquid electropolishing.



Improvement of the working environment

The DLyte process does not require any rubber tip, electrolytic bathing or brushing, but simply an abrasive preparation beforehand.

By reducing the traditional multi-step process, DLyte improves working conditions, and drastically reduces the toxicity of the current polishing.



From small to large production requirements

DLyte offers three equipment configurations to meet the different production capacity requirements of the dental industry. Each configuration has three versions, based on the materials that are to be treated.



DLyte **Desktop PRO**°

The most versatile ultra-compact dry electropolishing machine on the market.

TECHNICAL SPECIFICATIONS

 Capacity (per cycle)
 1 RPD

 Machine dimensions
 450 x 521 x 471 mm

 Machine weight
 33 kg

 Power
 1.7 kW

 Voltage
 230 V

 Metals
 CoCr & Titanium **



DLyte **Desktop Dental***

The first ultra-compact dry electropolishing machine for small dental laboratories.

TECHNICAL SPECIFICATIONS

Capacity (per cycle) 1 RPD
Machine dimensions 450 x 521 x 471 mm
Machine weight 33 kg
Power 1.7 kW
Voltage 230 V
Metals CoCr



DLyte1°

The most compact version.
This machine is designed for laboratories with low production requirements.

TECHNICAL SPECIFICATIONS

2 RPDs Capacity (per cycle) 510 x 1,150 x 690 mm Machine dimensions Support dimensions 510 x 690 x 680 mm 96 kg Machine weight Support weight 47 kg 2 kW ⊙ Power Voltage 220-240 V 4-5 bar Air pressure

All DLyte equipment comes with a one year warranty*.

^{**}and more metals with an additional pack.









^{*}Additional services.



DLyte 10°

Air pressure

This medium-sized system is easily integrated into a medium-sized laboratory.

TECHNICAL SPECIFICATIONS

Capacity (per cycle) 4 RPDs
Machine dimensions 820 x 1,280 x 680 mm
Support dimensions 820 x 680 x 680 mm
Machine weight 173,5 kg
Support weight 87 kg
Power 3 kW ©
Voltage 220-240 V

4-5 bar



DLyte100°

The solution for laboratories with large production requirements.

TECHNICAL SPECIFICATIONS

9 RPDs Capacity (per cycle) Machine dimensions 950 x 1,410 x 730mm Support dimensions 950 x 700 x 740mm Machine weight 217,5 kg 100 kg Support weight 5 kW ⊙ Power Voltage 220-240 V 4-5 bar Air pressure

Different versions for each configuration (only for DLyte1, DLyte10 and DLyte100).











